

## AT HOME WIRELESS SYSTEM

### Status Lights



The MXD-LTE speaks the system status each time the **RESET** button is pressed. The **Status** light displays the following light signals when operating:

LED Light	Description	Suggested Action
Fast Orange Flash	The first stage of the power-up sequence (cellular radio is powering on).	Wait for MXD-LTE to finish power-up sequence.
Steady Orange	The second stage of the power-up sequence (establishing network registration).	Wait for MXD-LTE to finish power-up sequence.
Steady Red	The unit is currently calling the central station.	Wait for MXD-LTE to finish call to the central station.
Single Green Flash	AC power not available, the MXD-LTE is running on battery.	Check A/C power connections.
Flashing Red	The unit is not receiving a cell signal.	<ol style="list-style-type: none"> <li>1. Move the unit to a better location. Press reset button to have the MXD-LTE indicate its number of bars. (2 or more bars is preferred)</li> <li>2. Press and hold the reset button until the status light begins to flash orange to hard reset the system and have it re-establish all connections.</li> <li>3. If none of the above work contact dealer support.</li> </ol>
Steady Green	The unit is properly connected.	No action needed.
Alternating Red/Green	In the process of deregistration and turning off the cellular radio.	Wait for MXD-LTE to finish its shutdown process.
Repeating Long Orange, short Green	A firmware update is being installed.	Wait for MXD-LTE to finish system update.
Alternating Red/Green/Orange	There was an error updating the firmware of the MXD-LTE. The MXD-LTE will need to be serviced. Contact your service provider for assistance.	The MXD-LTE will need to be serviced. Contact your service provider for assistance.